

# How to care for your eye protection goggles

**1**

Wipe the inside then outside of your goggles using a cleaning wipe or a clean cloth saturated with a neutral detergent.

**2**

Wipe the outside of the goggles using a TGA-approved disinfectant wipe or a clean cloth saturated with TGA-approved disinfectant.

**3**

Wipe the outside of the goggles with clean water to remove any residue.

**4**

Air dry.

**5**

Perform hand hygiene.

## Care tips

Label your goggles to ensure individual use.

To improve visibility, wipe the goggles with alcohol.

Store your goggles individually in a breathable container.



Scan to  
learn more